


# Item list

| <b>Pos</b> | <b>Art.nr</b> | <b>Code</b>   |
|------------|---------------|---|
| 101 1      | 002056        | Caster 75 mm  |
| 101 2      | 002057        | Caster 100 mm   |
| 102        | 003057        | Lazy lock knob  |
| 103 1      | 003053        | Frame with foot rest 75 mm caster                       |
| 103 2      | 003054        | Frame with foot rest 100 mm caster                      |
| 104        | 003056        | Front nut quick coupling                                |
| 105        | 003064        | Splash guard/abduction support                          |
| 106        | 003058        | Lock knob quick coupling                                |
| 107 1      | 003051        | Trunk support tube for chair with medium and large seat |
| 107 2      | 003052        | Trunk support tube for chair with small seat            |
| 108 1      | 003155        | Trunk support complete for large and medium seat        |
| 108 2      | 003156        | Trunk support complete for small seat                   |
| 109        | 003072        | Back harness  |
| 110        | 003055        | Push handle without back canvas                         |
| 111        | 003050        | Back canvas for push handle                             |
| 112        | 003078        | Safety strap  |
| 113 1      | 003060        | Small seat (2-5 years)                                  |
| 113 2      | 003160        | Medium seat (4-10 years)                                |
| 113 3      | 003260        | Large seat (8 years-60kg)                               |
| 114        | 003075        | Calf strap  |

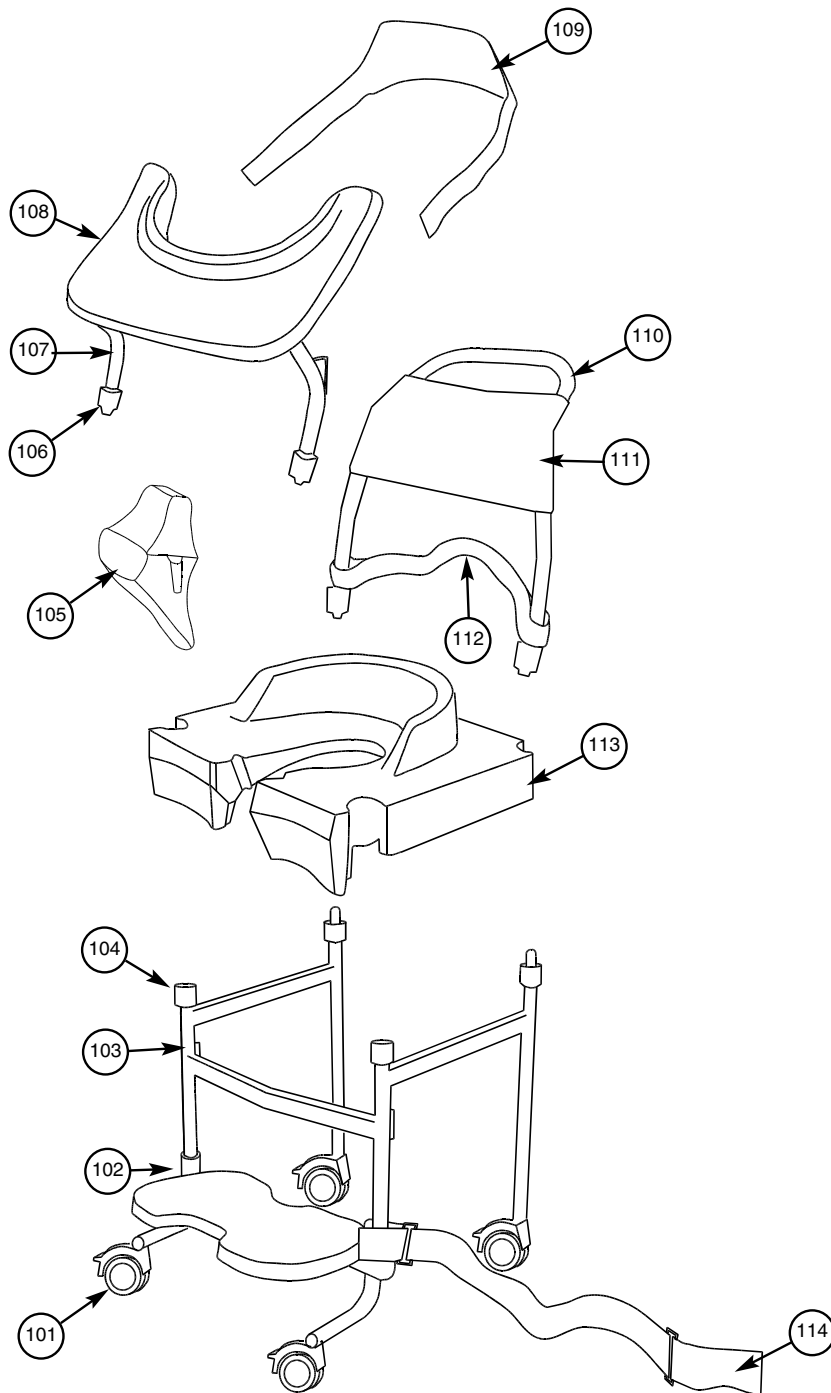

# Item list

| <b>Pos</b> | <b>Art.nr</b> | <b>Code</b>        |
|------------|---------------|--------------------|
| 201        | 003074        | H-harness          |
| 202        | 003081        | Pot                |
| 203        | 003090        | Transportation bag |

