


REAL 2001/2011 has exchangeable components, accessories and functions with adjustment options for individual requirements. The chair should only be used by the person and for the purpose for which it was originally intended. The chair may not be equipped with accessories or components other than those approved by Mercado Medic AB. Repairs and other technical adjustments may only be carried out by personnel authorised by Mercado Medic AB. Follow the instructions for care and cleaning. Read the instructions for use carefully before using the chair.

Backrest

Height adjustment: Loosen the knob (1). Adjust to desired height and then lock the knob (1).


Angle: Loosen the lever (2). Adjust to desired angle. Lock the lever (2).

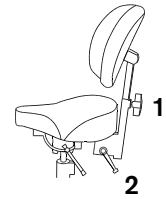

Sitting height


The height of the seat is adjusted using the lever (1) on one side of the seat (standard right). Lift the lever to change the height. The seat can not be seated when the height is adjusted.

To lower the seat: Be seated on the chair. Lift the lever (1), stop when the desired sitting height has been obtained.

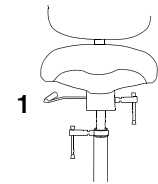


Rotation accessory (2011)

Chairs equipped with a rotation device can be manoeuvred using the handle (1). Turn anti-clockwise to release and clockwise to lock the device. Please note! When the rotation device is locked, the sitting height adjustment is locked too.

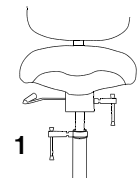

Sitting angle (2011)

Loosen the lever (1). Adjust to desired sitting angle and then lock the lever (1).

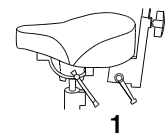

Cleaning the Fixlyftomat

The Fixlyftomat automatic device should be checked regularly and kept free of dust and grime. Move the seat base to the highest position. Clean with a cloth. Do not use water or any cleaning fluids. Lubricate the cylinder with a thin layer of Teflon or a silicon based lubricant. These are not as messy as mineral based oils.

Cleaning and maintenance

The chair should be wiped clean and kept free from dust and grime. The upholstery may be washed with foam cleaner. The seat, backrest and other upholstered parts are for functional reasons not made of sealed materials. For reasons of hygiene, the seat base and padding material should always be changed whenever the upholstery is changed.